

# Quick-Guide Implant Removal with new Placement Instrument



1



Open at the perforation and remove blister.

2



Open the blister.  
**Attention: this removes the sterile barrier.**

3



Remove the container and bring it in a upright position.

4



Push the lid sideways to remove it.

5



**Fix the implant by pressing lightly on the sides.**

6



Insert the new placement instrument without pressure and turn clockwise until the instrument slides into the connection.

Video Microcone



Video Quattrocone



**MEDENTIKA**  
A Straumann Group Brand

# Quick-Guide Implant Removal with new Placement Instrument



You hear or feel a click, the implant is secured on the placement instrument.



**Release the pressure** and remove the secured implant.



Insert the implant into the implant bed.




Turn the container upside down, where the closure screw is placed.

**Not included with QUATTROCONE30**



Unscrew the closure screw.

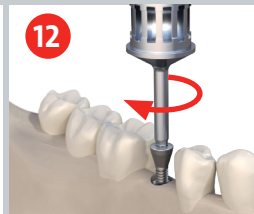

Place the closure screw in the implant.